

Tested Medical Grade Disinfectants for Venue 40 Console

To clean the monitor face:

Use the protective bag to wipe monitor face gently.

Or







Wipe the LCD monitor and Operator controls with the following cleaners:

- PDI Sani-Cloth Plus Germicidal Disposable Cloth (low Alcohol)
- PDI Super Sani-Cloth Germicidal Disposable Cloth (high Alcohol)
- PDI Sani-Cloth HB (Germicidal, Alcohol free)
- Sporidicin (Phenol)
- DisCide – recommended for clarity (63% isopropyl)
- Meliseptol Foam Pure

NOTE: Do NOT use a glass cleaner that has a hydrocarbon base (such as Benzene, Methyl Alcohol or Methyl Ethyl Ketone) on monitors with the filter (anti-glare shield). Hard rubbing will also damage the filter.

NOTE: When cleaning the screen, make sure not to scratch the LCD.

Venue 40 Transducer Care

			4C-SC	3S-SC	12L-SC	L8-18i-SC		
	Aquasonics 100	Gel	Parker Laboratory Inc.	OK	OK	OK	OK	-
	Scan	Gel	Parker Laboratory Inc.	OK	OK	OK	OK	-
	Sonogel	Gel	Sonogel Vertriebs GmbH	OK	OK	OK	OK	-
	Clear Image	Gel	Sonotech Inc.	OK	OK	⊗	⊗	-
	Kendall	Gel	Tyco/Healthcare	OK	⊗	⊗	⊗	-
	Wavelength	Gel	National Therapy Products Inc.	OK	⊗	⊗	⊗	-
	T-Spray	Quaternary Ammoniums	Pharmaceutical Innovations	⊗	OK	OK	OK	< 20 minutes
	T-Spray II	Quaternary Ammoniums	Pharmaceutical Innovations	OK	OK	OK	OK	< 20 minutes
	Asepti-wipes	8% IPA, <1% Quat	Ecolab Inc.	⊗	⊗	OK	⊗	< 30 minutes
	Asepti-wipes II	14.85% IPA	Ecolab Inc.	OK	OK	⊗	⊗	< 30 minutes
	CaviWipes	14.3% IPA	Metrex	OK	OK	⊗	⊗	< 30 minutes
	Septiwipes	Quaternary Ammoniums	Dr. Schumacher GmbH	OK	OK	OK	⊗	< 30 minutes
	Cleanisept wipes	Quaternary Ammoniums	Dr. Schumacher GmbH	OK	OK	OK	⊗	< 30 minutes
	Sani-Cloth HB	Quaternary Ammoniums	PDI	OK	OK	OK	OK	< 30 minutes
	PeraSafe	Peracetic Acid	Antec International	⊗	⊗	OK	OK	< 15 minutes
	Gigasept	Succindialdehyde, formaldehyde	Schülke & Mayr	⊗	⊗	OK	OK	< 50 minutes
	Cidex	Glutaraldehyde	Johnson & Johnson	OK	OK	OK	OK	< 50 minutes
	Cidex OPA	Ortho-phthalaldehyde	Johnson & Johnson	OK	OK	OK	OK	< 15 minutes
	Cidex Plus	Glutaraldehyde	Johnson & Johnson	OK	OK	OK	OK	< 25 minutes
	Gigasept AF	Quaternary Ammoniums	Schülke & Mayr	⊗	⊗	OK	OK	< 30 minutes
	Gigasept FF	Succindialdehyde	Schülke & Mayr	⊗	⊗	OK	OK	< 30 minutes
	Metricide 14	Glutaraldehyde	Metrex	OK	OK	OK	OK	< 50 minutes
	Ster Bac Blu	Quaternary Ammoniums	Ecolab Inc.	OK	OK	OK	OK	< 50 minutes
	Wavicide-01	Glutaraldehyde	MCC	OK	OK	OK	OK	< 50 minutes