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			6S	E8CS	IC5-9W-RS	12S-D	3Sc-RS		
			6S-RS						
			6S-D						
	<b>Aquasonics 100</b>	Gel	Parker Laboratory Inc.	OK	OK	OK	OK	OK	—
	<b>Clear Image</b>	Gel	Sonotech Inc.	OK	⊗	⊗	OK	OK	—
	<b>Kendall</b>	Gel	Tyco/Healthcare	OK	⊗	⊗	OK	OK	—
	<b>Scan</b>	Gel	Parker Laboratory Inc.	OK	OK	OK	OK	OK	—
	<b>Sonogel</b>	Gel	Sonogel Vertriebs GmbH	OK	OK	OK	OK	OK	—
	<b>Wavelength</b>	Gel	National Therapy Products Inc.	OK	⊗	⊗	OK	OK	—
	<b>T-Spray</b>	Quaternary Ammoniums	Pharmaceutical Innovations	⊗	OK	OK	⊗	⊗	< 20 min
	<b>T-Spray II</b>	Quaternary Ammoniums	Pharmaceutical Innovations	OK	OK	OK	OK	OK	< 20 min
	<b>Transeptic Spray</b>	Isopropyl Alcohol	Parker Laboratory Inc.	⊗	OK	OK	⊗	⊗	< 15 min
	<b>Asepti-Wipes II</b>	14.85%IPA	Ecolab Inc.	OK	⊗	⊗	OK	OK	< 30 min
	<b>CaviWipes</b>	14.3%IPA	Metrex	OK	⊗	⊗	OK	OK	< 30 min
	<b>Cleanisept wipes</b>	Quaternary Ammoniums	Dr.Schumacher GmbH	OK	⊗	⊗	OK	OK	< 30 min
	<b>Septiwipes</b>	Quaternary Ammoniums	Dr.Schumacher GmbH	OK	⊗	⊗	OK	OK	< 30 min
	<b>Sani-Cloth HB</b>	Quaternary Ammoniums	PDI	OK	OK	OK	OK	OK	< 30 min
	<b>PeraSafe</b>	Peracetic Acid	Antec International	⊗	OK	OK	⊗	⊗	< 15 min
	<b>Cidex</b>	Glutaraldehyde	Johnson & Johnson	OK	OK	OK	OK	OK	< 50 min
	<b>Cidex OPA</b>	Ortho-phthlaldehyde	Johnson & Johnson	OK	OK	OK	OK	OK	< 15 min
	<b>Cidex Plus</b>	Glutaraldehyde	Johnson & Johnson	OK	OK	OK	OK	OK	< 25 min
	<b>Gigasept AF</b>	Quaternary Ammoniums	Schülke & Mayr	⊗	OK	OK	⊗	⊗	< 30 min
	<b>Gigasept FF</b>	Succindialdehyde	Schülke & Mayr	⊗	OK	OK	⊗	⊗	< 30 min
	<b>Hibitane</b>	Chlorhexidine	Regent Medical	⊗	OK	OK	⊗	⊗	< 30 min
	<b>Metricide 14</b>	Glutaraldehyde	Metrex	OK	OK	OK	OK	OK	< 50 min
	<b>Wavicide-01</b>	Glutaraldehyde	MMC	OK	OK	OK	OK	OK	< 50 min